EoloStent

Prótesis laríngea veterinaria

ES

MEDICAL ALERT TAG

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EN

MEDICAL ALERT TAG

Safety and Clinical Management Sheet

Intended Use

Temporary device designed to maintain laryngeal patency in cases of stenosis of various etiologies (laryngeal paralysis, laryngeal collapse, or other obstructive emergencies).

Warnings and Precautions

- Caution during **intubation** with the stent: use an endotracheal tube with a diameter smaller than the stent; do not displace the prosthesis.
- For veterinary use only.
- Single-use product.
- Do not resterilize or reuse.

Possible Adverse Effects


- Allergic reactions (silicone, associated medication).
- Migration or fracture of the device.
- Formation of granulation tissue (mass effect).
- Local infection.
- Mucus accumulation.

Identification of an Adverse Event with EoloStent™

- Inspiratory dyspnea
- Nausea
- Sialorrhea
- Others

Recommended Examination

Direct visualization of the larynx under deep sedation or induction.

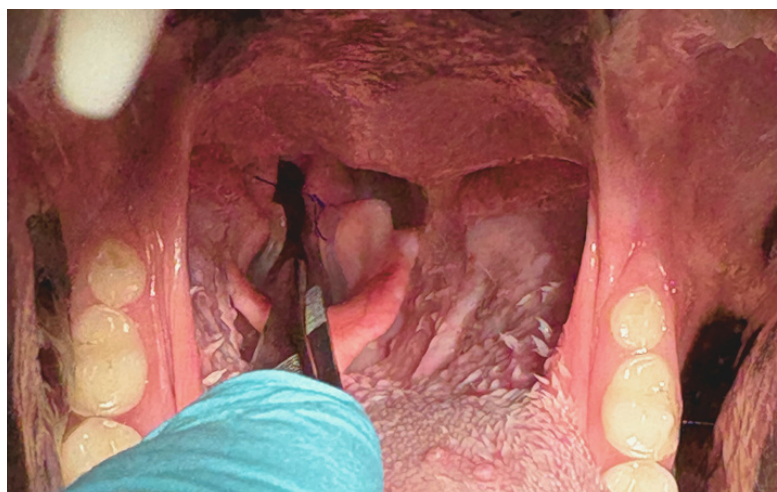Management Recommendations

- Device migration: remove and reposition.
- Inflammation: administer corticosteroids and reassess. Do not remove during the inflammatory phase.
- Excessive mucus accumulation / infection: identify the cause and replace the prosthesis. If necessary, prescribe antibiotic therapy.

Stent Removal

- Cut the fixation sutures attaching the stent to the arytenoid cartilages.
- Gently pull with forceps; compression of the cranial ends may facilitate removal, but is not essential.

Repositioning

- Use of long instruments: essential.
- Gently compress the prosthesis with curved atraumatic forceps.
- Reinsert under direct visualization until the caudal ring is positioned behind the laryngeal cartilages.
- Ensure that the cranial wings rest on the cuneiform processes.
- Secure both sides with non-absorbable sutures.

Recommendations

- Place the suture from outside to inside of the laryngeal lumen, at each cuneiform process.
- Use a non-absorbable monofilament suture, ½ circle < 3/0.

Download the full product datasheet at

 www.epiicvet.com